

# BAS-200slm

## Bio-Acoustic Simulator Sound Level Meter



### Monitor Background Noise During Testing

If you're testing in locations where it's inconvenient to have a sound booth, then you need to prove that tests were completed within legal limits for ambient noise.

The BAS-200slm offers all the dependability of the BAS-200 with a sound level meter. It is a dual-function instrument that creates a legal record of the daily calibration check and of compliance with OSHA ambient noise regulation as required by 29 CFR 1910.95.

The BAS-200slm meets the ANSI requirements for both a Type 2 Octave Band Filter and a Type 2 Sound Level Meter so your compliance is documented.

Because the BAS-200slm is powered by the Benson Medical Instruments audiometer, there is no worry about battery life. Connecting the response switch to the BAS-200slm returns the system to subject testing.

### Get Started Today

Contact your Benson Medical Instruments dealer for a no-obligation demonstration.

- ✓ Quick and accurate daily calibration check as required by OSHA
- ✓ Ensures compliance with OSHA ambient noise level regulations
- ✓ Power is supplied from audiometer
- ✓ Place on tabletop or mount on wall of test booth
- ✓ No need to remove from system when performing subject testing
- ✓ Meets ANSI standards for both sound level meters and octave band filters
- ✓ Documents compliance with OSHA regulations
- ✓ States in reports that ambient noise levels were not exceeded
- ✓ Stores ambient noise levels with audiogram data
- ✓ Supports OSHA, ANSI, or user-defined maximum

**SUPERIOR  
AUDIOMETRICS**  
Phone: 303-517-9788  
Info@superioraudiometrics.com

**BENSON MEDICAL INSTRUMENTS**

## Assured Compliance

OSHA regulation 29 CFR 1910.95 Appendix D requires that the ambient noise in the audiometric testing room not exceed certain levels. The BAS-200slm provides iron-clad documentation that the test was performed within the limits set by OSHA.

For each audiogram the following data is stored:

- Ambient noise levels for each octave band
- Maximum permissible ambient noise levels for each octave band
- BAS-200slm serial number and calibration date

The audiometer will re-present stimuli when ambient noise levels are exceeded. Reports will state that the audiogram was performed within the OSHA ambient noise limits, providing a legal record for every audiogram.

## True Sound Level Meter

Benson Medical Instruments is the only company to seamlessly integrate an ANSI Type 2 sound level meter into the audiometric testing system so that you are assured of continuous compliance. Only readings from an ANSI Type 2 Sound Level Meter will meet the standards set by OSHA. Other so-called 'monitors' do not meet these strict criteria because they do not comply with ANSI standards.

## Bio-Acoustic Simulator

A bio-acoustic simulator is included for a quick and accurate daily calibration check as required by OSHA. Because the simulator does not use batteries, you are assured of the most stable and accurate reference for comparison.

## System Requirements

When the CCA-100mini is used with two stations, one or two BAS-200slm's may be used.

When the CCA-200mini is used, the total number of instruments connected (CCA-200mini's and BAS-200slm's) must not exceed sixteen.

## Simulator Specifications

Indicators: power, left response, right response  
Test Frequencies: 500 Hz, 1, 2, 3, 4, 6, 8 kHz  
Response Threshold: 65 dB HL nominal  
Power: supplied by Benson Medical Instruments audiometer

## Sound Level Meter Specifications

Meets or exceeds ANSI S1.4-1983 Type 2 Sound Level Meter standard.

## Octave Band Filter Specifications

Meets or exceeds ANSI S1.11-1986 Type 2 Order 3 Octave Band Filter standard.

Meets or exceeds ANSI S1-11-1971 Class II Octave Band Filter standard.

## Maximum Permissible Noise Levels

CCA-100mini:

OSHA 29 CFR 1910.95 App. D only

CCA-200mini:

OSHA 29 CFR 1910.95 App. D, or ANSI

S3.1-1999 (ears covered), or user-settable

in 0.5 dB steps between 21.0 and 62.0 dB SPL for 125, 250, 500 Hz, 1, 2, 4, and 8 kHz octave bands